# beurer IH 21

- |   |  |
|---|--|
| <b>D Inhalator</b><br>Gebrauchsanweisung .....      | <b>TR Inhalatör</b><br>Kullanım kılavuzu .....             |
| <b>GB Nebuliser</b><br>Instructions for use .....   | <b>RUS Ингалятор</b><br>Инструкция по<br>применению .....  |
| <b>F Inhalateur</b><br>Mode d'emploi .....          | <b>PL Inhalator</b><br>Instrukcja obsługi .....            |
| <b>E Inhalador</b><br>Manual de instrucciones ..... | <b>ELECTROMAGNETIC COMPATIBILITY<br/>INFORMATION</b> ..... |
| <b>I Inhalatore</b><br>Istruzioni per l'uso .....   | <b>70-72</b>   |

## Inhalt

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## Contents

1. Getting to know your instrument .....	11
2. Signs and symbols .....	11
3. Safety information and warnings .....	11
4. Description of device and accessories .....	13
5. Initial use .....	13
6. Operation .....	14
7. Changing the filter .....	15
8. Cleaning and disinfection .....	15
9. Disposal .....	16
10. Troubleshooting .....	17
11. Technical specifications .....	17

## Included in delivery

For details of the delivery scope, see description of the device and accessories on page 13.

- Nebuliser
- Atomiser
- Compressed air hose
- Mouthpiece
- Adult mask
- Child mask
- Nosepiece
- Spare filter
- These instructions for use

## 1. Getting to know your instrument

### Dear customer

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

With kind regards

Your Beurer team

### Area of application

This nebuliser is a device used for the nebulisation of liquids and liquid medication (aerosols) and for the treatment of the upper and lower respiratory tract.

The nebulisation and inhalation of medication prescribed or recommended by a doctor can prevent against diseases of the respiratory tract, treat the side effects of these disorders and accelerate healing.

For more information on possible applications, consult your doctor or pharmacist.

The device is suitable for inhalation at home. Medicines should only be inhaled when instructed by a doctor. Inhalation should be performed in a calm and relaxed atmosphere. Inhale slowly and deeply to enable the medicine to reach the small bronchi deep into the lungs. Exhale normally.

Once prepared, the device is suitable for reuse. The preparation includes the replacement of all accessory parts including atomiser and air filter, as well as disinfection of the surfaces of the device with a standard disinfectant.

## 2. Signs and symbols

The following symbols appear in these instructions.

-  **Warning** Warning instruction indicating a risk of injury or damage to health.
-  **Important** Safety note indicating possible damage to the unit/accessory.
-  **Note** Note on important information.

The following symbols are used on the packaging and on the nameplate of the device and accessories.

	Type BF applied part
	Read the instructions for use
	Device in protection class 2
	Manufacturer
I	On
O	Off
SN	Serial number
	Protect from moisture

## 3. Safety information and warnings

### Warning

- Before use, ensure that there is no visible damage to the unit or accessories. When in doubt, do not use the unit and contact your dealer or the customer service address provided.
- Using the unit is not a substitute for consultation with or treatment from a physician. Whenever you have any pain or illness, always contact your physician first.

- If you have health concerns of any kind, contact your general practitioner!
- When using the atomiser, always follow the basic hygiene procedures.
- Always follow your doctor's instructions regarding the type of medicine to be used and the dose, frequency, and duration of inhalation.
- Do not use the device in conjunction with ventilation systems.
- Only use medicines that have been prescribed or recommended by your doctor or pharmacist.
- If the unit does not function correctly or if you begin to feel unwell or feel pain, stop using the unit immediately.
- During use, hold the unit away from the eyes, as some nebulised medicines may cause damage to the eyes.
- Do not operate the device in the presence of flammable gases or if there is a high oxygen concentration.
- This device is not intended to be used by children or persons with restricted physical, sensory (e.g. insensitivity to pain) or mental abilities or persons lacking in the required experience or knowledge for safe operation of the device, unless supervised or instructed by a person responsible for their safety.
- Store the device and its accessories out of the reach of children to prevent the risk of accidents and choking as a result of swallowing small parts or packaging material!
- Do not use attachments that are not recommended by the manufacturer.
- Connect the unit only to the mains voltage listed on the type plate.
- Do not dip the unit into water and do not use it in wet rooms. Do not allow any liquid to penetrate the unit.
- Protect the unit from strong impacts.
- Never touch the mains cable with wet hands, this can cause an electric shock.
- Do not pull the mains plug from the socket by the cord.
- Do not pinch or bend the mains cable, pull it over sharp objects, or allow it to hang freely, and protect it against sources of heat.
- If the mains cable or housing is damaged, contact our customer service department or your local dealer.
- Opening the unit carries a risk of electric shock. Disconnection from the mains supply is only guaranteed if the mains plug has been removed from the socket.
- Do not use the unit if it has fallen or been dropped on the floor, been exposed to extreme humidity or is otherwise damaged. In case of doubt, contact our customer service department or your local dealer.
- The IH 21 nebuliser must only be used with suitable Beurer atomisers and with the appropriate Beurer

accessories. The use of other atomisers and accessories can impair therapeutic efficiency and may damage the device.

### Important

- Power cuts, sudden faults or other unfavourable conditions may render the unit unusable. You are therefore recommended to keep a spare device or replacement medicine (as agreed with the doctor).
- If adapters or extensions are required, these must comply with the applicable safety regulations. The electrical capacity and the maximum capacity specified on the adapter must not be exceeded.
- The unit and the mains cable must be stored away from sources of heat.
- Do not use the unit in a room in which a spray has previously been used. If this is the case, ventilate the room before commencing treatment.
- Do not allow any objects to enter or obstruct the ventilation openings.
- Do not use the device if you notice an unusual noise.
- For hygiene reasons, each user must use his own accessories.
- Always disconnect from the mains after use.
- Store the device in a location protected against environmental influences. The device must be stored in the specified ambient conditions.

### FUSE

- The device contains an overload protection fuse. This must only be changed by authorised specialist personnel.
- The device also features a thermal fuse that switches off the nebuliser in the event of overheating. Should this occur, proceed as follows:
  - Switch off the appliance.
  - Disconnect from the mains.
  - Wait for a minimum of 30 minutes until the device has completely cooled down.
  - Then switch the device on again, and observe whether it makes any abnormal noises. If there are no unusual noises, you can use the device again. Otherwise, contact your customer service team.

### General notes

#### Important

- Use the unit only:
  - On humans,
  - for its intended purpose (aerosol inhalation) and in the manner described in these instructions for use.
- **Any improper use can be dangerous!**
- In acute emergencies, first aid takes priority.
- Apart from the medication, use only distilled water or a saline solution in the unit. Other liquids may cause defects to the nebuliser or atomiser.

- This unit is not intended for commercial or clinical use, but only for individual, private household use!

## Before using the unit for the first time

### Important

- Before you use the unit for the first time, remove all packaging materials.
- Protect the unit against dust, dirt and moisture. Never cover the unit during operation.
- Do not use the unit in very dusty environments.
- Switch the unit off immediately if it is defective or malfunctioning.
- The manufacturer shall not be held liable for damage or injuries caused by improper or incorrect use.


## Repairs

### Note

- Never open or attempt to repair the unit yourself, as otherwise proper function is no longer guaranteed. Failure to observe this regulation shall void the warranty.
- If you need to have the unit repaired, contact customer service or an authorised dealer.


## 4. Description of device and accessories

### Overview of the nebuliser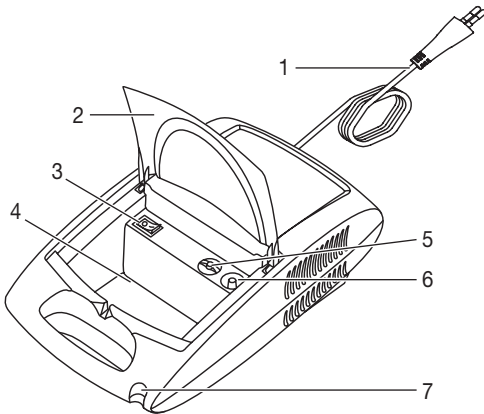

- 1 Mains cable
- 2 Lid
- 3 On/off switch
- 4 Compartment for accessories
- 5 Filter cap with filter
- 6 Hose connection
- 7 Holder for atomiser

## Overview of atomiser and accessories

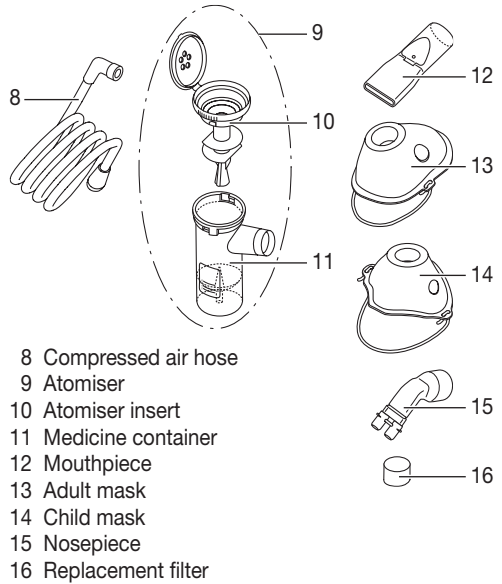- 8 Compressed air hose
- 9 Atomiser
- 10 Atomiser insert
- 11 Medicine container
- 12 Mouthpiece
- 13 Adult mask
- 14 Child mask
- 15 Nosepiece
- 16 Replacement filter

## 5. Initial use

### Setup

Remove the unit from the packaging. Place it on a flat surface.


Ensure that the ventilation slits are free.

Fold open the lid [2] to access the accessories.


### Before using the unit for the first time

#### Note

- The atomiser and accessories must be cleaned and disinfected before they are used for the first time. For information about this, see “Cleaning and disinfection” on page 15.
- Attach the compressed-air hose [8] at the bottom of the medicine container [11].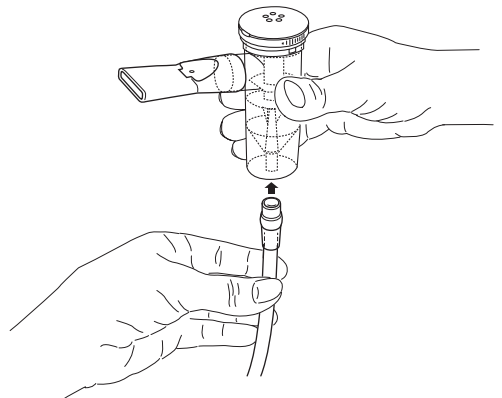

- Connect the other end of the hose [8] to the hose connector [6] on the nebuliser by lightly turning it.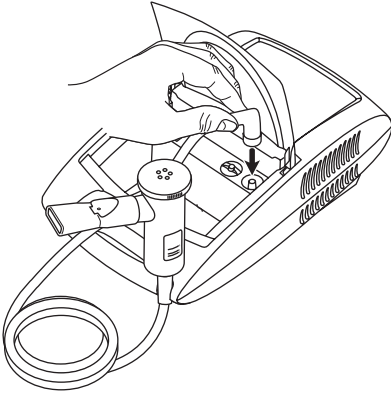

## Mains connection

Connect the unit only to the mains voltage listed on the type plate.

- Insert the mains cable plug [1] into a suitable socket.
- Push the plug completely into the mains socket.

### Note

- Ensure that there is a socket near to where the unit will stand.
- Lay the mains cable in such a way that no one can trip over it.
- To disconnect the nebuliser from the mains after use, first switch off the device and then remove the plug from the socket.


## 6. Operation

### Caution

- For hygiene reasons, the atomiser [9] and the accessories must be cleaned after each treatment and disinfected after the last treatment of the day.
- If several different medicines are to be inhaled in succession during therapeutic use, the atomiser [9] must be rinsed under warm tap water after each use. For information about this, see “Cleaning and disinfection” on page 15.
- Follow the instructions for changing the filter contained in these instructions.
- Before each use, check that the hose connections on the nebuliser [6] and the atomiser [9] are tight.
- Before use, check that the device is functioning correctly. To do this, switch on the nebuliser (including the connected atomiser, but without any medication) for a short time. If the atomiser [9] expels air, the device is functioning correctly.


## 1. Assembling the atomiser

- Insert the atomiser insert [10] into the medication container [11] and turn clockwise.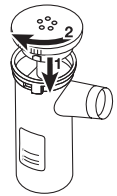

## 2. Filling the atomiser

- Open the lid of the atomiser [9].
- Fill with an isotonic saline solution or pour the medication directly into the atomiser [9]. Do not overfill! The maximum recommended fill level is 8 ml!
- Use medication only on the advice of a physician, and check the appropriate inhalation duration and quantity for your needs.
- If the required volume of medication is less than 2 ml, top up this quantity to at least 4 ml using an isotonic saline solution. Viscous medicines may also require dilution. Always follow the instructions of your doctor.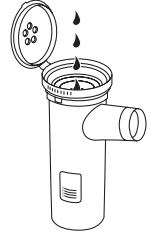

## 3. Closing the atomiser

- Close the lid of the atomiser [9].

## 4. Connecting the accessories to the atomiser

- Connect the required accessory part (mouthpiece [12], adult mask [13], child mask [14] or nosepiece [15]) to the atomiser [9].

### Note

The most effective method of nebulisation is by using the mouthpiece. Nebulisation using a mask is only recommended if it is not possible to use a mouthpiece (e.g. for children who are not yet able to inhale medication using a mouthpiece).

- Before commencing treatment, pull the atomiser upwards out of its holder [7].
- Start the nebuliser using the on/off switch [3].
- A flow of spray from the atomiser indicates that the device is functioning properly.

## 5. Treatment

- Inhale the nebulised medicine deeply.

### Note

During treatment, hold the atomiser straight (vertical), otherwise the atomisation does not work and correct function cannot be guaranteed.

## Important

Essential oils, cough medicines, solutions designed for gargling, and drops for application to the skin or for use in steam baths are wholly unsuitable for inhalation using a nebuliser. These substances are often highly viscous and can impair the correct function of the device and hence affect the effectiveness of its application in the long term.

In the case of oversensitivity of the bronchial system, medicines containing essential oils can sometimes cause acute bronchospasm (a sudden, cramp-like tightening of the bronchi accompanied by breathlessness). Always ask your doctor or pharmacist for advice!

## 6. End of inhalation

If the spray is only output sporadically or if the noise changes during inhalation, you can end the treatment.

- After treatment, turn off the nebuliser at the switch [3] and disconnect it from the mains.
- After use, replace the atomiser [9] in its holder [7].

## 7. Clean the device

For information about this, see “Cleaning and disinfection” on page 15.

## 7. Changing the filter

Check the filter regularly (e.g. after every tenth use). Replace the used filter if it is very dirty or blocked, but at the latest after 30 days. If the filter has become wet, it must also be replaced with a new one.

## Caution

- Do not try to clean and re-use the used filter!
- Only use original filters supplied by the manufacturer. Otherwise, your nebuliser may become damaged or effective therapy can no longer be guaranteed.
- Never operate the device without a filter.

To replace the filter, proceed as follows:


## Caution

- First switch off the unit and unplug it.
- Allow the unit to cool down.

1. Remove filter cap [5] upwards.

## Note

If the filter remains in the unit when the cap is removed, remove the filter from the unit using tweezers or similar.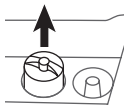

2. Replace the filter cap [5] containing the new filter.
3. Check that the filter cap is firmly in place.

# 8. Cleaning and disinfection

## Atomiser and accessories

## Warning

Observe the following hygiene instructions in order to avoid any health risks.


- The atomiser [9] and accessories are intended for multiple use. Please note that cleaning requirements differ according to the different application areas.

## Notes:

- The atomiser and accessories must not be mechanically cleaned with brushes or similar as this can cause irreparable damage and the success of the treatment can no longer be guaranteed.
- For additional requirements regarding the necessary hygienic preparations (hand washing, handling of medicines or inhalation solutions) in high-risk groups (e.g. cystic fibrosis patients), contact your doctor.
- Ensure that all parts are properly dried following all cleaning and disinfection. Residual moisture or water droplets can result in an increased risk of bacterial growth.

## Preparation

- Directly following each treatment, all parts of the atomiser [9] and any accessories used must be cleaned of medicine residues and contamination.
- To do this, dismantle the atomiser [9] into its individual parts.
- Remove the mouthpiece [12] from the atomiser.
- Dismantle the atomiser by turning the atomiser insert [10] anticlockwise against the medication container [11] and pull upwards.
- The nebuliser is later reassembled in the reverse order.

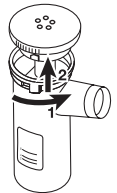## Cleaning

## Important

Before you clean the unit, always switch it off, unplug it and let it cool off.

The **atomiser** and any **accessories** used, such as the mouthpiece, mask, nozzle attachment, etc., must be cleaned with hot water.

Carefully dry all components using a soft cloth. Once all components are fully dry, reassemble the atomiser and place the components in a dry, sealed container or carry out disinfection.

When cleaning, ensure that all residues are removed. Never use any substances that may be poisonous when in contact with the skin or mucous membranes, or when swallowed or inhaled.

Clean the **device** using a damp cloth, to which you can apply a little detergent if necessary.

Do not use abrasive detergents and never immerse the unit in water.

### Important

- Ensure that no water penetrates inside the unit!
- Do not put the device or accessories in a dishwasher.
- Do not touch the unit with wet hands while it is plugged in; do not allow any water to be sprayed onto the unit. The unit must be operated only when it is completely dry.
- Do not spray any liquids into the ventilation slots. Any liquid that penetrates into the unit can damage the electrical parts or other components of the nebuliser and impair the function of the device.

### Condensation, care of the hose

Depending on the environmental conditions, condensation may collect in the hose. To prevent against bacterial growth and to ensure effective therapeutic use, this moisture must always be removed. To remove moisture from the hose, proceed as follows:

- Remove the hose [8] from the atomiser [9].
- Leave the hose plugged into the nebuliser [6].
- Run the nebuliser until all the moisture has been displaced by the flow of air.
- In the event of significant contamination, replace the hose.

### Disinfection

Follow the steps listed below carefully in order to disinfect your atomiser and the accessories. It is advisable to disinfect the individual components after the last time they are used each day at the latest.

You can disinfect the IH 21 and its accessories in two ways: using vinegar or boiling water.

First, clean the atomiser and accessories as described in the “Cleaning” section. Then continue with the disinfection.

#### Disinfection using vinegar

- Use a vinegar solution consisting of ¼ vinegar and ¾ distilled water. Make sure that the volume is sufficient to fully submerge the parts, such as the atomiser, mask and mouthpiece, in the solution.
- Leave the parts in the vinegar solution for 30 minutes.
- Then rinse the atomiser and the individual parts such as the mask and mouthpiece with water.

#### Disinfection with boiling water

- The atomiser should be taken apart and placed together with the mouthpiece and nose-piece in boiling hot water for at least 10 minutes. Avoid the parts coming into contact with the hot base of the pot.

- You can also disinfect the atomiser, the mouthpiece and the nose-piece with a standard vaporizer. Observe the manufacturer’s instructions for use for the vaporizer when doing this.

### Note

The face masks and air hose must not be placed in hot water!

- Carefully dry all components using a soft cloth.
- We recommend disinfecting the masks using a retail disinfectant.
- Once all components are fully dry, reassemble the atomiser and place the components in a dry, sealed container.

### Note

Please ensure that the parts are completely dried after cleaning, otherwise the risk of bacterial growth is increased.

### Drying

- Place the individual components on a dry, clean and absorbent surface and allow to dry completely (for at least 4 hours).

### Material resistance

- As with all plastic components, frequent use and cleaning of the atomiser and the accessories can lead to a certain amount of wear. Over time, this can alter the aerosol properties and may eventually affect the efficiency of therapy. We therefore recommend that you replace the atomiser once a year.
- When choosing a cleaning or disinfecting agent, note the following: Only use mild cleaning or disinfectant agents in the quantities recommended by the manufacturer.

### Warehousing

- Do not store the nebuliser in a damp atmosphere (e.g. in the bathroom) or transport it together with damp objects.
- Store and transport the nebuliser away from direct sunlight.

### Storage

- The accessories can be safely stored in the accessories compartment [4]. Store the device in a dry place, if possible in its original packaging.

## 9. Disposal

In the interest of environmental protection, do not dispose of the device in household waste.

Please dispose of the device in accordance with EC Directive – **WEEE** (Waste Electrical and Electronic Equipment).

If you have any queries, please contact the appropriate local authorities.

## 10. Troubleshooting

Problem/Question	Possible Cause/Remedy
The atomiser produces little or no aerosol.	1. Too much or too little medicine in the atomiser. Minimum: 0.5 ml, Maximum: 8 ml.
	2. Check that the nozzles are not blocked. If necessary, clean the nozzle (e.g. by rinsing). Use the atomiser again. <b>CAUTION:</b> Only poke implements through the fine holes from the underside of the nozzle.
	3. The atomiser is not held in an upright position.
	4. The medicine solution is unsuitable for nebulising (e.g. too thick). The medicine solution should be specified by the doctor.
Output is too low.	Hose kinked, filter blocked, too much inhalation solution.
Which medicines are suitable for inhalation?	<b>Consult your doctor.</b> Essentially all medicines that are suitable and approved for device-based inhalation can be inhaled.
Some inhalation solution remains in the atomiser.	This is normal and occurs for technical reasons. Stop inhalation if you hear a notable difference in the noise of the atomiser.
What special steps should be taken for babies and children?	1. In babies, the mask should cover the mouth and nose to guarantee effective inhalation. 2. In children, the mask should also cover the mouth and nose. Nebulisation next to a sleeping person is not suitable because insufficient medicine reaches the lungs. <b>Note:</b> Children should only use the device with help and under supervision of an adult. Never leave a child alone with the nebuliser.
Inhalation with the mask takes longer.	This is for technical reasons. Less medicine is inhaled per breath through the mask holes than using the mouthpiece. The aerosol is mixed with ambient air through the holes in the mask.


Problem/Question	Possible Cause/Remedy
Why do I need to replace the atomiser at regular intervals?	There are two reasons for this: 1. To guarantee a therapeutically effective particle spectrum, the nozzle holes should not exceed a certain diameter. Due to the thermal and mechanical stresses, the plastic is subjected to a certain amount of wear. The atomiser [9] is particularly sensitive. This can also cause changes to the droplet composition of the aerosol, which has a direct effect on the effectiveness of the treatment. 2. You are also recommended to replace the atomiser on a regular basis for hygiene reasons.
Does each person need their own atomiser?	This is absolutely necessary for hygiene reasons.

## 11. Technical specifications

Dimensions (WxHxD)	300 x 180 x 100 mm
Weight	1.9 kg
Operating pressure	approx. 1.2 bar
Atomiser fill volume	max. 8 ml
Medicine flow	>0.2 ml/min.
Sound pressure	<60 dBA (acc. to DIN EN 13544-1 section 26)
Mains connection	230 V ~, 50 Hz; 180 VA
Housing material	ABS
Operating conditions	Temperature: +10 °C – +40 °C Relative humidity: 10% – 95%
Storage and transport conditions	Temperature: -20 °C – +70 °C Relative humidity: 10% – 95% Atmospheric pressure: 500 – 1060 hPa

Subject to technical modifications.

### Particle size diagram

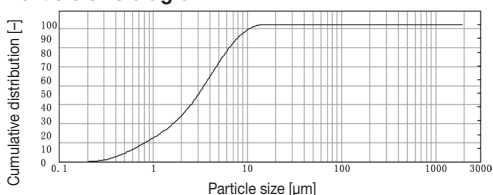All measurements were obtained with a sodium chloride solution using a laser diffraction method.

This diagram may not be applicable for suspensions or highly viscous medicines. More information can be obtained from the relevant medicine manufacturer.

### Spare parts

Designation	Material	REF
Nebulisation stopper	PP	162.819
Yearpack (contains Mouth-piece, Adult mask, Child mask, Atomiser, Compressed air hose, Filter, Nosepiece)	PP/PVC	601.03

### Note

If the unit is used outside of the specifications, proper function is no longer guaranteed!

We reserve the right to make technical changes to improve and further develop the product.

This device and its accessories comply with European standards EN60601-1 and EN60601-1-2, as well as EN13544-1, and is subject to special safety measures in terms of electromagnetic tolerance. Note that portable and mobile RF communication equipment can affect this unit. More details can be requested from the stated Customer Service address or found at the end of the instructions for use. The unit conforms to the requirements of the European Directive for Medical Products 93/42/EC, the MPG (German Medical units Act).

### ELECTROMAGNETIC COMPATIBILITY

- The device complies with current specifications with regard to electromagnetic compatibility and is suitable for use in all premises, including those designated for private residential purposes. The radio frequency emissions of the device are extremely low and in all probability do not cause any interference with other devices in the proximity.
- It is recommended that you do not place the device on top of or close to other devices. Should you notice any interference with other electrical devices, move the unit or connect it to a different socket.
- Radio equipment may affect the operation of this device.

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## ELECTROMAGNETIC COMPATIBILITY INFORMATION

### Guidance and manufacturer's declaration – electromagnetic emissions

The IH21 is intended for use in the electromagnetic environment specified below. The customer or the user of the IH21 should assure that it is used in such an environment.

Emmissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The IH21 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The IH21 is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

### Guidance and manufacturer's declaration – electromagnetic immunity

The IH21 is intended for use in the electromagnetic environment specified below. The customer or the user of the IH21 should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Power frequency (50Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

### Guidance and manufacturer's declaration – electromagnetic immunity

The IH21 is intended for use in the electromagnetic environment specified below. The customer or the user of the IH21 should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	$3 V_{\text{rms}}$ 150 kHz to 80 MHz	$3 V_{\text{rms}}$	Portable and mobile RF communications equipment should be used no closer to any part of the IH21, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.  Recommended separation distance $d = 1.2 \sqrt{P}$
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	$d = 1.2 \sqrt{P}$ 80 MHz to 800 MHz  $d = 2.3 \sqrt{P}$ 800 MHz to 2.5 GHz  where $P$ is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and $d$ is the recommended separation distance in metres (m).  Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, <sup>a</sup> should be less than the compliance level in each frequency range. <sup>b</sup>  Interference may occur in the vicinity of equipment marked with the following symbol: 

**NOTE 1** At 80 MHz and 800 MHz, the higher frequency range applies.

**NOTE 2** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

**a** Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the IH21 is used exceeds the applicable RF compliance level above, the IH21 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the IH21.

**b** Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.

## Recommended separation distances between portable and mobile RF communications equipment and the IH21

The IH21 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the IH21 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the IH21 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz	80 MHz to 800 MHz	800 MHz to 2.5 GHz
	$d = 1.2 \sqrt{P}$	$d = 1.2 \sqrt{P}$	$d = 2.3 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

**NOTE 1** At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

**NOTE 2** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.