EC REP

Medeco BV  
Alexander Flemingstraat 2  
NL-3261 MA Oud-Beijerland  
The Netherlands

Medexel Co., Ltd.  
536-65, Geumgwang-Ri,  
Geumgwang-Myeon,  
Ansong-Si, Gyeonggi-Do, Korea


Code	Product	Quantity	Dimensions
5065	soft fine plus	100 + 10st/pcs	0.23 mm (32G) x 4 mm
5064	soft fine plus	100 + 10st/pcs	0.23 mm (32G) x 5 mm
5066	soft fine plus	100 + 10st/pcs	0.25 mm (31G) x 5 mm
5060	soft fine plus	100 + 10 st/pcs	0.25 mm (31G) x 6 mm
5061	soft fine plus	100 + 10 st/pcs	0.25 mm (31G) x 8 mm
5067	soft fine plus	100 + 10st/pcs	0.30 mm (30G) x 8 mm
5062	soft fine plus	100 + 10 st/pcs	0.33 mm (29G) x 10 mm
5063	soft fine plus	100 + 10 st/pcs	0.33 mm (29G) x 12 mm

0\_040401

# soft fine® plus

- Pennaald
- Pen Needle
- Pen-Nadelen
- Penkanyler
- Penī nōel
- Aguja para pluma de insuline
- Kynäneula
- Aiguille pour stylo à insuline
- Tú inzulinollhoz
- Ago per penna
- Insulino švirkušto adata
- Pildspalvveida injektorā adata
- Kanyler for injeksjonspenn
- Pennkanyl
- Iglā do wstrzykiwaczy
- Игла для шприц-ручки
- Kalem ĩgnesi

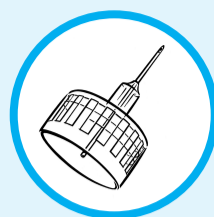## NEDERLANDS

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## ENGLISH

soft fine plus is a pen needle that can be used with most common pen systems as detailed on the packaging. Please consult the doctor in charge of your treatment regarding the appropriate needle type and length and how to use the pensystem in combination with the needle.

### Attention!

- Avoid infection. The sterility of the pen needle can only be guaranteed until the use-by date and if the blister packaging is undamaged. Use the needle once only.
- In order to ensure delivery of the correct dosage, check the flow through the needle with the help of the functionality test detailed in the instructions supplied with your pen system.
- Don't change the angle of the pen system after piercing the skin with the needle, as this can cause the needle to bend.
- After using the pen needle take care for proper removal by using a needle container.

### Safety first!

- Store the needles in a cool, dry place. Keep away from heat and damp conditions.
- In order to avoid hardening of the skin, change the injection site regularly in consultation with your doctor or carer.
- If you notice that fluid drips out of the needle after it has been removed from the skin, leave the needle in place for an additional 10 seconds after future injections, to ensure that the dose is complete.
- After using the needle, replace the outer cap carefully. Please take care not to touch anything with the tip of the needle, in order to avoid infection.
- Always use a needle once only. When used repeatedly, it becomes blunt or bent which may cause minor injuries. Repeated use causes the effects of the silicon layer to diminish.

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