# ACCU-CHEK<sup>®</sup>

## *Rapid-D Link*

INFUSION SET

CE 0123

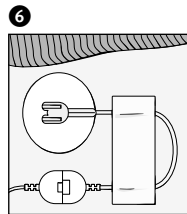
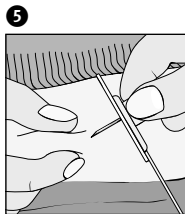
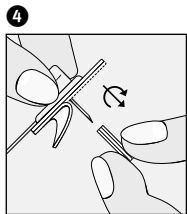
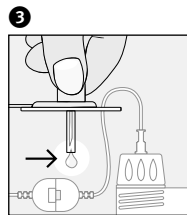
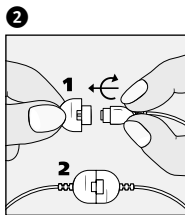
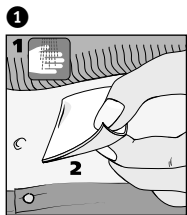
ACCU-CHEK is a trademark of Roche.

Disetronic Medical Systems AG  
CH-3401 Burgdorf, Switzerland  
[www.disetronic.com](http://www.disetronic.com)

04537220001/V02/06.06

<b>Instructions for use</b>	<b>4</b>	<b>Brugsanvisning</b>	<b>104</b>
<b>Gebrauchsanweisung</b>	<b>12</b>	<b>Οδηγίες χρήσης</b>	<b>112</b>
<b>Mode d'emploi</b>	<b>20</b>	<b>Kasutusjuhised</b>	<b>122</b>
<b>Istruzioni per l'uso</b>	<b>30</b>	<b>הוראות שימוש</b>	<b>130</b>
<b>Instrucciones de uso</b>	<b>38</b>	<b>Upute za uporabu</b>	<b>138</b>
<b>Gebruiksaanwijzing</b>	<b>46</b>	<b>Használati útmutató</b>	<b>146</b>
<b>Instruções de uso</b>	<b>54</b>	<b>Naudojimo instrukcijos</b>	<b>154</b>
<b>Návod k použití</b>	<b>62</b>	<b>Lietošanas instrukcija</b>	<b>162</b>
<b>Instrukcja użytkowania</b>	<b>70</b>	<b>Instrucțiuni de utilizare</b>	<b>170</b>
<b>Bruksanvisning</b>	<b>78</b>	<b>Инструкции по использованию</b>	<b>178</b>
<b>Bruksanvisning</b>	<b>86</b>	<b>Návod na používanie</b>	<b>188</b>
<b>Käyttöohjeet</b>	<b>94</b>	<b>Navodila za uporabo</b>	<b>196</b>
		<b>Uputstvo za upotrebu</b>	<b>204</b>
		<b>Kullanım talimatları</b>	<b>212</b>
		<b>使用说明</b>	<b>220</b>
		<b>使用說明</b>	<b>228</b>


<b>Instructions for use</b>	<b>en</b>
<b>Gebrauchsanweisung</b>	<b>de</b>
<b>Mode d'emploi</b>	<b>fr</b>
<b>Istruzioni per l'uso</b>	<b>it</b>
<b>Instrucciones de uso</b>	<b>es</b>
<b>Gebruiksaanwijzing</b>	<b>nl</b>
<b>Instruções de uso</b>	<b>pt</b>
<b>Návod k použití</b>	<b>cs</b>
<b>Instrukcja użytkowania</b>	<b>pl</b>
<b>Bruksanvisning</b>	<b>sv</b>
<b>Bruksanvisning</b>	<b>no</b>
<b>Käyttöohjeet</b>	<b>fi</b>

# ACCU-CHEK<sup>®</sup>
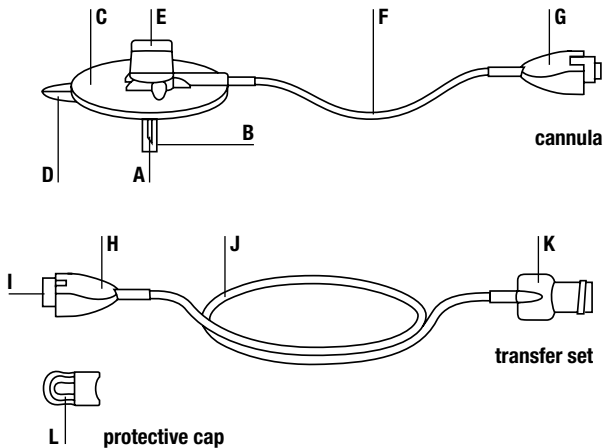
## Rapid-D Link

### INFUSION SET

## Description

Accu-Chek Rapid-D Link is an infusion set for the subcutaneous infusion of insulin administered with microdosage insulin pumps.

### Contents of the cannula package for the Accu-Chek Rapid-D Link

- A steel cannula
- B protective cover
- C self-adhesive
- D adhesive backing
- E disc holder
- F tubing
- G connecting piece of cannula

### Contents of the transfer set package for the Accu-Chek Rapid-D Link

- H connecting piece of transfer set
- I connector needle
- J tubing
- K luer-lock

### Contents of the protective cap package for the Accu-Chek Rapid-D Link

- L protective cap

## **⚠ WARNINGS**

Read the instructions carefully before use. It is recommended that you consult your insulin pump manual for information regarding pump therapy.

For first time use, setup must be performed in the presence of a health care professional.

- This device remains sterile until the package is opened or damaged. Do not use if the sterile package has been opened or damaged.
- Protect the product from excessive humidity, sunlight and heat. Store at room temperature.
- Before inserting the Accu-Chek Rapid-D Link, prepare the infusion site in accordance with the instructions of your healthcare professional.
- Discard the set after each use. Do not clean or re-sterilize.
- The Accu-Chek Rapid-D Link is for subcutaneous use only.
- Always tighten the luer-lock firmly into the pump adapter. If this is not done, danger of leakage can result, due to a loose connection between the infusion set and the pump. Avoid using any tools to screw the luer-lock connector into place. Otherwise damage could result to the luer-lock connector.
- Do not leave air bubbles in the Accu-Chek Rapid-D Link. Prime thoroughly
- Never insert the Accu-Chek Rapid-D Link before priming, or attempt to release blockages while it is in use, otherwise there is a danger of an uncontrolled quantity of insulin being infused into the body.
- Never use a steel cannula for more than 48 hours (2 days). Never use the transfer set for more than 6 days. When changing the cartridge, make sure that the transfer-set is completely primed before connecting to the cannula.

- If the self-adhesive does not stick firmly to the skin, e.g. after a shower or in the case of heavy sweating, affix the Accu-Chek Rapid-D Link with a transparent adhesive plaster. If the Accu-Chek Rapid-D Link has slipped out of the skin, use a new infusion set.
- If the infusion site becomes inflamed, replace your Accu-Chek Rapid-D Link and use a new site.
- Do not put disinfectants, perfumes, deodorants or insect repellent on your Accu-Chek Rapid-D Link as these may affect the stability of the set.
- Dispose of the infusion set safely, so that nobody can prick or injure themselves on it.
- If you have to disconnect your Accu-Chek Rapid-D Link, pay particular attention to hygiene and use the protective cap.
- Avoid excessive stress on the infusion site and on the Accu-Chek Rapid-D Link. Pay attention when carrying heavy weights.
- Check blood glucose level 1–3 hours after inserting your Accu-Chek Rapid-D Link and inspect the infusion site on a regular basis. Do not insert a new infusion set just prior to bedtime. Check blood glucose frequently (check with your physician).
- If blood glucose level gets unexplainably too high, or an «occlusion» alarm occurs, check for clogs and leaks. If in doubt, change your Accu-Chek Rapid-D Link.

## Insert Accu-Chek Rapid-D Link

### **Picture 1**

Choose a site away from the waistline, bones, scar tissue, belly button and recently used infusion sites. Clean hands and infusion site according to instructions from your healthcare professional. Allow the injection site to dry before inserting the Accu-Chek Rapid-D Link.

### **Picture 2**

Remove the cannula and the transfer set from the sterile packaging. Hold the two halves of the coupling so that they form a cross. Fit the parts directly into one another and connect them in place by turning for one quarter of a turn in a clockwise direction until they click noticeably and audibly into place.

### **⚠ WARNING**

If the two halves of the coupling do not click noticeably and audibly together, use a new cannula and a new transfer set.

### **Picture 3**

Screw the luer-lock (K) firmly into the screw thread of the pump adapter. Make sure there is a cartridge containing sufficient insulin in the cartridge compartment. Place the pump upright and fill the Accu-Chek Rapid-D Link in accordance with the pump manufacturer's operating instructions. Prime the tubing (F, J) until the medication emerges at the tip of the steel cannula (A). Do not leave air bubbles in either the cartridge or in the tubing of the Accu-Chek Rapid-D Link.

**Fill volumes of the complete set:**

30 cm/12"	approx. 7 units U 100 insulin	approx. 70 $\mu$ l
60 cm/24"	approx. 10 units U 100 insulin	approx. 100 $\mu$ l
80 cm/31"	approx. 13 units U 100 insulin	approx. 130 $\mu$ l
110 cm/43"	approx. 18 units U 100 insulin	approx. 180 $\mu$ l

**Picture 4**

Hold the Accu-Chek Rapid-D Link firmly between thumb and forefinger at the disc holder (E). With the other hand, remove the adhesive backing (D) from the self-adhesive (C) and twist the protective cover (B) off the steel cannula (A).

**Picture 5**

With your free hand, pinch together a bit of subcutaneous tissue at the disinfected site. Insert the Accu-Chek Rapid-D Link under the skin at a perpendicular angle. Press the self-adhesive (C) firmly onto the skin in order to ensure secure seating of Accu-Chek Rapid-D Link. Bend the disc holder (E) slightly sideways and remove it. In doing this, make sure that the self-adhesive comes to lay flat on the skin.

** WARNING**

Change the head-set every 2 days (max. 48 hours). Otherwise you risk insulin resistance or inflammation of the infusion site.

Never use the transfer-set for more than 6 days.

**Picture 6**

Form a loop to relieve stress and secure it into place with a strip of adhesive plaster. Start distribution through the pump.

## Disconnect Accu-Chek Rapid-D Link

### **WARNING**

Please consult with your health care professional on how to compensate for any missed insulin while you are disconnected. Carefully monitor blood glucose level while disconnected from the pump and after you reconnect.

#### **Step 1**

Turn off the pump.

#### **Step 2**

Grasp the two halves of the coupling and turn for a quarter of a turn in an anticlockwise direction. Pull the two halves of the coupling away from each other, proceeding with caution.

#### **Step 3**

Cover both halves of the coupling with the protective caps (L).

## Reconnect Accu-Chek Rapid-D Link

#### **Step 1**

Remove the protective caps from the two halves of the coupling.

#### **Step 2**

Start distribution through the pump. Fill the transfer set until the medication starts trickling out of the tip of the connector needle (I).

**Step 3**

Fit the two halves of the coupling together carefully and turn for one quarter of a turn in a clockwise direction until they click audibly into place (see Picture 2).

**Distributed in United Kingdom by:**

Roche Diagnostics Ltd.

Bell Lane

Lewes

**UK**-East Sussex BN7 1LG

Phone +44 1273 480444

Fax +44 1273 480266

lewes.insulinpumps@roche.com

www.accu-chek.co.uk

**Distributed in New Zealand by:**

Roche Diagnostics N.Z. Ltd

P.O. Box 62-089

15 Rakino Way

Mt. Wellington

**NZ**-Auckland

Phone +64 9 276 4157

Fax +64 9 276 8917

info@accu-chek.co.nz

www.accuchek.co.nz

**Distributed in Australia by:**

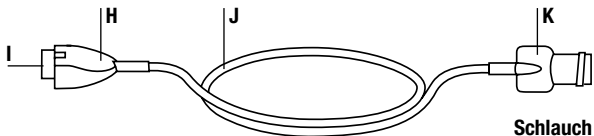
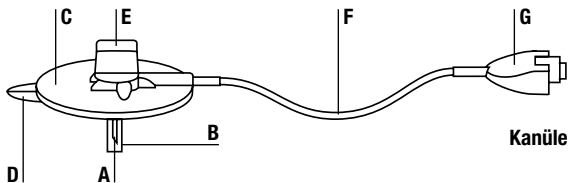
Roche Diagnostics Australia Pty. Ltd.

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