Rx only

Made in Switzerland

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ACCU-CHEK, ACCU-CHEK FLEXLINK, ACCU-CHEK LINKASSIST and ACCU-CHEK ULTRAFLEX are trademarks of Roche.

ACCU-CHEK, ACCU-CHEK FLEXLINK, ACCU-CHEK LINKASSIST y ACCU-CHEK ULTRAFLEX son marcas registradas de Roche.

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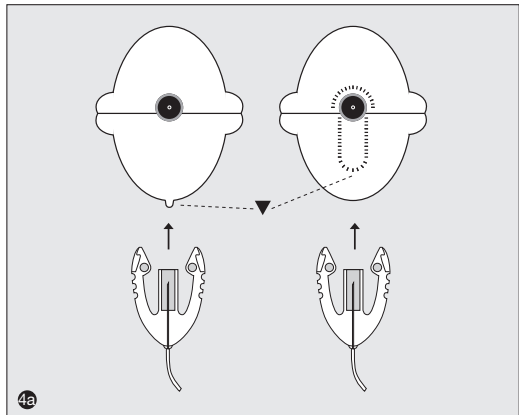
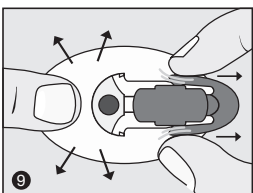
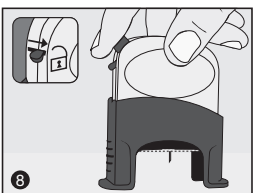
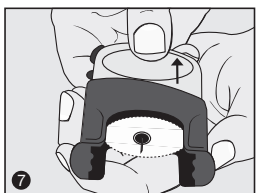
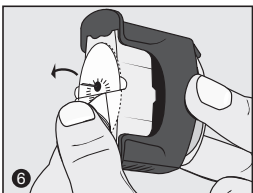
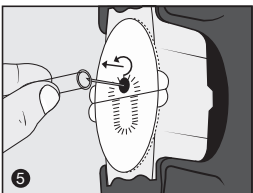
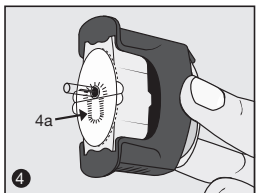
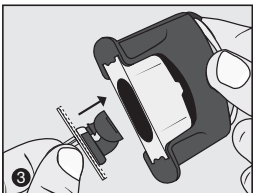
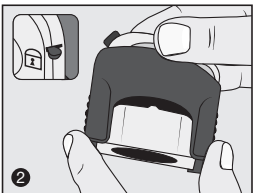
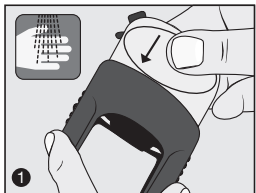
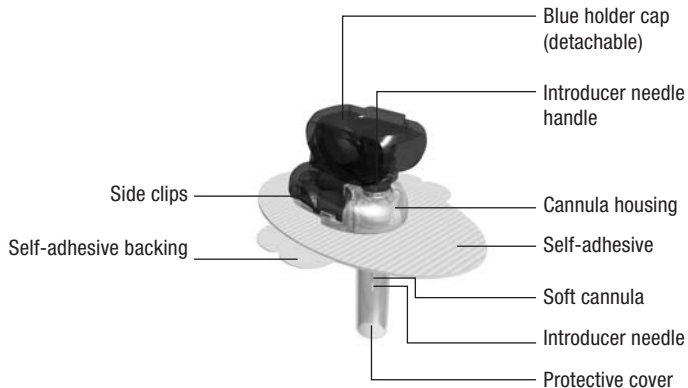
INSERTION DEVICE

en **Instructions for use**

Accu-Chek LinkAssist is an infusion set insertion device, which is intended specifically for placement of the Accu-Chek FlexLink (in USA/Canada: Accu-Chek Ultraflex) infusion set. It permits an easy, quick and virtually painless insertion of the cannula into the subcutaneous fatty tissue for the subcutaneous infusion of insulin.

Accu-Chek FlexLink Infusion Set (USA/Canada: Accu-Chek Ultraflex)

CONTRAINDICATIONS

- The Accu-Chek LinkAssist insertion device is solely intended for use with the Accu-Chek FlexLink (in USA/Canada: Accu-Chek Ultraflex) infusion set. A safe and smooth operation of the Accu-Chek LinkAssist insertion device cannot be guaranteed if any other type of infusion set is used.
- The Accu-Chek LinkAssist insertion device is exclusively for use on one person. It must not be used on several people.

WARNING

Carefully read these instructions for use and the instructions for use for the Accu-Chek FlexLink (in USA/Canada: Accu-Chek Ultraflex) infusion set before using the insertion device. The sequence of the steps of application described in these instructions must always be observed. Failure to follow the instructions for use may cause pain and injury.

Preparation

- Before using the insertion device, carefully read the instructions for use for the Accu-Chek FlexLink (in USA/Canada: Accu-Chek Ultraflex).
- Please wash your hands before use. Choose a site away from waistline, bones, scar tissue, belly button and recent infusion sites. Prepare site according to the instructions from your health care professional. Allow the infusion site to dry before the soft cannula is inserted.
- Open the undamaged packaging of the infusion set and remove the infusion set.

WARNINGS

- Do not use any infusion set if the introducer needle has been damaged or bent.
- The infusion set remains sterile until the package is opened or damaged. Do not use if the sterile package has been opened or damaged.

How to use the Accu-Chek LinkAssist insertion device to insert the Accu-Chek FlexLink (in USA/Canada: Accu-Chek Ultraflex) infusion set

Please also refer to the pictures on the cover flap.

- 1 Push the tensioning element of the insertion device into its base.
- 2 The insertion device is now pretensioned. This step “locks” the safety catch in place, and automatically makes the insertion device secure.
- 3 Hold the infusion set by its protective cover between the thumb and the forefinger and with the blue holder cap pointing upward, insert it from below into the opening of the insertion device until you feel it lock into place.
- 4 Correct placement of the infusion set.

Note: The marking ▼ on the self adhesive backing directs you to the location of the tube connection.

WARNING

If an infusion set is in the insertion device, never look into the opening of the device. Do not use the insertion device near the face or eyes.

- 5 Carefully remove the transparent protective cover of the infusion set by turning and pulling at the same time.
- 6 Remove both pieces of the self-adhesive backing from the infusion set self-adhesive.
- 7 Hold the base of the insertion device firmly in one hand and with the other hand pull the tensioning element out and up as far as it will go.

⚠ WARNING

Only release the insertion device once it is positioned on the skin.

- 8** Holding the tensioning element, position the insertion device onto the prepared infusion site. Activate the insertion device by sliding the safety catch out of the “lock” position. Press the release button to introduce the cannula into the skin. The insertion device can now be removed from the infusion site and set aside.

⚠ WARNING

In case the infusion set gets stuck in the insertion device, carefully push the tensioning element back into the base without pointing the insertion device toward any body part or somebody else. Continue with step 7. Make sure not to compress the base during application.

- 9** Check the correct position of the self-adhesive and press this self-adhesive layer smoothly and firmly onto the skin in order to ensure secure positioning of the infusion set.

⚠ WARNING

If the infusion set is incorrectly positioned, then a new infusion set must be used.

- Hold the self-adhesive in place. Carefully press the side clips of the blue holder cap together and remove it from the cannula housing.
- To remove the introducer needle, hold the cannula housing firmly in place on the body. Place one finger on the top of the cannula housing and withdraw the introducer needle by its handle in a straight line. Discard your introducer needle and infusion set after use according to local regulations.

⚠ WARNING

Use caution when covering the steel needle in order to minimize the risk of an accidental stick.

Connection to pump

Connect the tubing to the cannula housing and the insulin pump according to the instructions for use for the Accu-Chek FlexLink (in USA/Canada: Accu-Chek Ultraflex) infusion set and the instructions for use for the particular insulin pump.

⚠ WARNINGS

- Check blood glucose level 1 – 3 hours after inserting your infusion set and inspect the infusion site on a regular basis. Do not insert a new soft cannula just prior to bedtime. Check blood glucose frequently (check with physician).

- Ensure clean application throughout.
- If you dispose of the Accu-Chek LinkAssist, please place device in trash can or other suitable container.
- Dispose of used infusion sets safely.
- Never re-use an infusion set.
- The infusion set should only be introduced into those sites of the body recommended by your healthcare professional.

Cleaning and Care

- If needed, wipe the outside of the Accu-Chek LinkAssist insertion device with a cloth moistened beforehand with water, a mild hand soap or a standard disinfectant (e.g. 70 % ethyl alcohol).
- Do not dip the insertion device into the cleaning fluid.
- Dry the insertion device thoroughly with a soft, clean cloth.

Correct storage

- The insertion device should not be stored in a pretensioned condition.
The tensioning element should be completely outside the base part and the release button triggered (see Figure 1).
- Do not store the insertion device in extreme temperatures (for example in a hot car).

Warranty

Your Accu-Chek LinkAssist is warranted against defects in material and workmanship for 18 months from the date of purchase.

US

For detailed warranty and service information please refer to the Warranty section at the end of this manual.

Distributed in the USA by:

Roche Health Solutions Inc.
11800 Exit 5 Parkway, Suite 120
Fishers, IN 46037, **USA**
Toll Free 1-800-280-7801
ACCU-CHEK Customer Care Service Center:
1-800-688-4578
www.accu-chek.com

Distributed in the United Kingdom by:

Roche Diagnostics Limited
Charles Avenue, Burgess Hill
West Sussex, RH15 9RY, **United Kingdom**
Accu-Chek Pump Care Line¹⁾:
UK Freephone number: 0800 731 22 91
ROI Freephone number: 1 800 88 23 51
¹⁾ calls may be recorded for training purposes
burgesshill.insulinpumps@roche.com
www.accu-chek.co.uk
www.accu-chek.ie

Distributed in Australia by:

Roche Diagnostics Australia Pty. Ltd.
31 Victoria Avenue, Castle Hill
NSW 2154, **Australia**
ABN 29 003 001 205
Pump Support: 1800 633 457
Orders: 1800 802 409
www.accu-chek.com.au
For training videos, visit
www.youtube.com/accuchek

Distributed in New Zealand by:

Roche Diagnostics NZ Ltd
15 Rakino Way, Mt. Wellington 1641
PO Box 62-089, Sylvia Park 1644
Auckland
New Zealand
Pump Hotline: 0800 696 696
Accu-Chek Enquiry Line: 0800 80 22 99
www.accu-chek.co.nz

Distributed in South Africa by:

Roche Products (Pty) Ltd.
Diagnostics Division
P.O. Box 1927, Randburg 2125
9 Will Scarlet Road, Ferndale
Randburg, **South Africa**
Accu-Chek Customer Care Centre:
080-DIABETES (Dial 080-34-22-38-37)
www.diabetes.co.za

Distributed in Canada by:

Roche Diagnostics
201 Armand-Frappier Blvd.
Laval, Quebec, H7V 4A2, **Canada**
Accu-Chek Pump Support: 1-800-688-4578
(toll free)
www.accu-chek.ca

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ar

Accu-Chek LinkAssist Limited 18-Month Warranty

Roche Insulin Delivery Systems Inc. (“Roche”) warrants to the original purchaser of the Accu-Chek LinkAssist insertion device that your Accu-Chek LinkAssist insertion device will be free from defects in materials and workmanship for 18 months from the date of purchase. If, during this 18-month period, the insertion device does not work properly because of a defect in materials or workmanship, Roche will replace it with a new Accu-Chek LinkAssist insertion device or equivalent product free of charge. The warranty on the replacement insertion device will expire on the date of the original warranty expiration or 90 days after the shipment of a new insertion device, whichever period is longer. The purchaser’s exclusive remedy with respect to the Accu-Chek LinkAssist insertion device shall be replacement.

This warranty does not apply to the performance of an insertion device that has been damaged by accident or has been altered, misused, tampered with, or abused in any way.

THE ABOVE WARRANTY IS EXCLUSIVE OF ALL OTHER WARRANTIES, AND ROCHE MAKES NO OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING WITHOUT LIMITATION, THE IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE BE LIABLE TO THE PURCHASER OR ANY OTHER PERSON FOR ANY INCIDENTAL, CONSEQUENTIAL, INDIRECT, SPECIAL, OR PUNITIVE DAMAGES ARISING FROM OR IN ANY WAY CONNECTED WITH THE PURCHASE OR OPERATION OF THE ACCU-CHEK

LINKASSIST INSERTION DEVICE OR ITS PARTS. NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, IF ANY IS IMPLIED FROM THE SALE OF THE INSERTION DEVICE, SHALL EXTEND FOR A LONGER DURATION THAN EIGHTEEN MONTHS FROM THE DATE OF PURCHASE.

Some states do not allow limitations on how long an implied warranty will last or the exclusion of incidental or consequential damages, so the above limitation and exclusion may not apply to you. This warranty gives you specific legal rights, which vary from state to state.

Warranty and Service Instructions

All requests for return of the Accu-Chek LinkAssist insertion device under the above warranty must be made to Roche Insulin Delivery Systems Inc., Customer Care Service Center, 11800 Exit 5 Parkway, Suite 120, Fishers, IN 46037, USA. You will be mailed a return authorization label, which must be affixed to your carton for shipping the system to Roche. Cartons received without this label will be returned to you at your expense.

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