beurer IH 50

D Inhalator Gebrauchsanweisung	2-11	TR Inhalatör Kullanım kılavuzu	50-57
GB Nebuliser Instructions for use	12-20	RUS Ингалятор Инструкция по применению	58-67
F Inhalateur Mode d'emploi	21-30	PL Inhalator Instrukcja obsługi	68-76
E Inhalador Manual de instrucciones	31-40	Electromagnetic Compatibility Information	77-78
I Inhalatore Istruzioni per l'uso	41-49		

Inhalt

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Contents

1. Getting to know your instrument.....	12
2. Signs and symbols.....	12
3. Safety information and warnings.....	12
4. Useful information on the device.....	14
5. Description of device and accessories	15
6. Initial use	15
7. Operation.....	16
8. Cleaning and disinfection.....	17
9. Technical specifications	19
10. Disposal.....	20

Included in delivery

For details of the delivery scope, see description of the device and accessories on page 15.

- Nebuliser and accessories
- Mesh atomiser with medicine container
- Mouthpiece
- Connecting piece
- Adult mask
- Child mask
- Power pack
- These instructions for use
- Storage bag

1. Getting to know your instrument

Dear customer

thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, baby and air.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

With kind regards
Your Beurer team

Area of application

This nebuliser is a mesh atomiser for treating the upper and lower respiratory tract.

The nebulisation and inhalation of medication prescribed or recommended by a doctor can prevent against diseases of the respiratory tract, treat the side effects of these disorders and accelerate healing.

For more information on possible applications, consult your doctor or pharmacist.

The device is suitable for inhalation at home. Medicines should only be inhaled when instructed by a doctor. Inhalation should be performed in a calm and relaxed atmosphere. Inhale slowly and deeply to enable the medicine to reach the small bronchi deep into the lungs. Exhale normally.

Once prepared, the device is suitable for reuse. Reconditioning includes the replacement of all accessories including the mesh atomiser, as well as the disinfection of the surface of the device using a standard disinfectant.

2. Signs and symbols

The following symbols appear in these instructions.

-  **Warning** Warning instruction indicating a risk of injury or damage to health.
-  **Important** Safety note indicating possible damage to the unit/accessory.
-  **Note** Note on important information.

The following symbols are used on the packaging and on the nameplate of the device and accessories.

	Type BF applied part
	Follow instructions for use
	Manufacturer
	On/Off
SN	Serial number
	Device in protection class 2
	Do not use outdoors
IP 22	Protected against foreign objects equal to or greater than 12.5 mm in diameter and against drops of water falling at up to 15° from vertical

3. Safety information and warnings

Warning

- Before use, ensure that there is no visible damage to the unit or accessories. When in doubt, do not use the unit and contact your dealer or the customer service address provided.
- Using the unit is not a substitute for consultation with or treatment from a physician. Whenever you have any pain or illness, always contact your physician first.

- If you have health concerns of any kind, contact your general practitioner!
- When using the nebuliser, always follow the basic hygiene procedures.
- Always follow your doctor's instructions regarding the type of medicine to be used and the dose, frequency, and duration of inhalation.
- Only use medicines that have been prescribed or recommended by your doctor or pharmacist.
- If the unit does not function correctly or if you begin to feel unwell or feel pain, stop using the unit immediately.
- During use, hold the unit away from the eyes, as some nebulised medicines may cause damage to the eyes.
- Do not operate the unit in the presence of flammable gases.
- Do not use the device near electromagnetic transmitters.
- This device is not intended to be used by children or persons with restricted physical, sensory (e.g. insensitivity to pain) or mental abilities or persons lacking in the required experience or knowledge for safe operation of the device, unless supervised or instructed by a person responsible for their safety.
- Keep children away from packaging materials (risk of suffocation).
- Do not use attachments that are not recommended by the manufacturer.
- Do not dip the unit into water and do not use it in wet rooms. Do not allow any liquid to penetrate the unit.
- Protect the unit from strong impacts.
- Do not use the unit if it has fallen or been dropped on the floor, been exposed to extreme humidity or is otherwise damaged. In case of doubt, contact our customer service department or your local dealer.
- The IH 50 must only be operated using the appropriate Beurer accessories. The use of other accessories can impair therapeutic efficiency and may damage the device.
- Remove the batteries from the device if you will not be using it for an extended period. Leaking batteries may damage the device.

Important

- Power cuts, sudden faults or other unfavourable conditions may render the unit unusable. You are therefore recommended to keep a spare device or replacement medicine (as agreed with the doctor).
- The device must be stored away from sources of heat.
- Do not use the unit in a room in which a spray has previously been used. If this is the case, ventilate the room before commencing treatment.

- For reasons of hygiene, each user should use their own set of accessories (mesh atomiser, mask and mouthpiece).
- Store the device in a location protected against environmental influences. The device must be stored in the specified ambient conditions.
- Do not use any liquids that have a viscosity (thickness) greater than 3, as these may damage the mesh irreparably.
- Do not use any medicines in powder form (even if dissolved).
- Do not shake the atomiser, as this may cause liquid to leak, and the device may become damaged.

Power pack

Important

- Do not use any power pack other than the one supplied.
- Ensure that the mains socket cannot come into contact with liquids.
- If adapters or extensions are required, these must comply with the applicable safety regulations. The electrical capacity and the maximum capacity specified on the adapter must not be exceeded.
- After using the power pack, unplug it from the socket.
- Connect the unit only to the mains voltage listed on the type plate.

General notes

Important

- Use the unit only:
 - On humans,
 - for its intended purpose (aerosol inhalation) and in the manner described in these instructions for use.
- **Any improper use can be dangerous!**
- In acute emergencies, first aid takes priority.
- Apart from medicines, use only a saline solution.
- This unit is not intended for commercial or clinical use, but only for individual, private household use!

Before using the unit for the first time

Important

- Before you use the unit for the first time, remove all packaging materials.
- Protect the unit against dust, dirt and moisture.
- Do not use the unit in very dusty environments.
- Before use, ensure the device and accessories are at room temperature.
- Switch the unit off immediately if it is defective or malfunctioning.
- The manufacturer shall not be held liable for damage or injuries caused by improper or incorrect use.

Notes on handling batteries

- Keep batteries out of the reach of children. Children can put batteries in their mouths and swallow them. This can cause severe harm to their health. In this case, consult a physician immediately!
- Normal batteries must not be recharged, heated or thrown into open flame (**risk of explosion**).
- Leaking batteries may damage the device. If you do not intend to use the device for prolonged periods, remove the batteries from the battery compartment.

Warning

Leaking or damaged batteries can cause burns if they touch the skin. When removing such batteries, wear suitable protective gloves.

Repairs

Note

- Never open or attempt to repair the unit yourself, as otherwise proper function is no longer guaranteed. Failure to observe this regulation shall void the warranty.
- If you need to have the unit repaired, contact customer service or an authorised dealer.

4. Useful information on the device

Accessories

The safe functioning of the device can only be guaranteed if the accessories recommended by the manufacturer are used.

Important

Protecting against leaks

When filling the medicine container with medicine, ensure that you do not exceed the maximum mark (8 ml). The recommended filling amount is between 2 and 8 ml.

During use, you can tilt the device up to a maximum angle of 45° in any direction, without impairing its nebulising function or the success of the treatment. However, spraying only takes place if the substance to be sprayed is in contact with the mesh. If it is not, spraying stops automatically.

However, please try to hold the device as upright as possible.

Automatic switch-off

The device has an automatic switch-off function. To prevent damage to the mesh, the device switches off automatically when the medicine/liquid has been almost entirely used up.

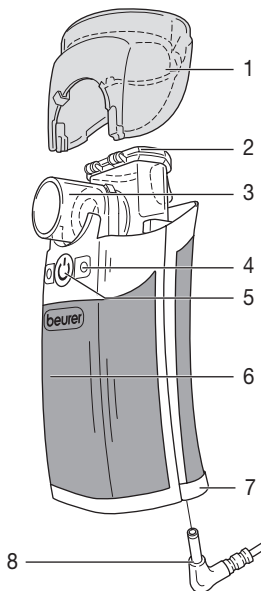
Do not use the device if the medicine container or water tank is empty.

The device switches off automatically if the substance to be sprayed is no longer in contact with the mesh.


5. Description of device and accessories

Overview

- 1 Cover
- 2 Medicine container
- 3 Mesh atomiser
- 4 Operation LEDs:
blue: device ready
orange: change batteries
- 5 On/off button
- 6 Casing
- 7 Battery compartment
- 8 Power pack connector

Overview of accessories

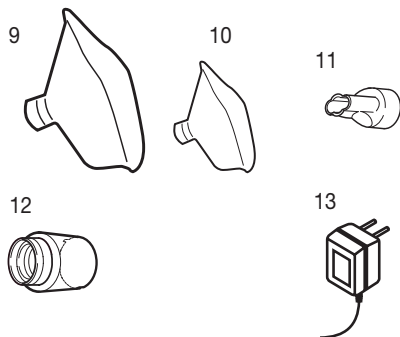- 9 Adult mask
 - 10 Child mask
 - 11 Mouthpiece
 - 12 Connecting piece
 - 13 Power pack
- Storage bag

6. Initial use

Before using the unit for the first time

i Note

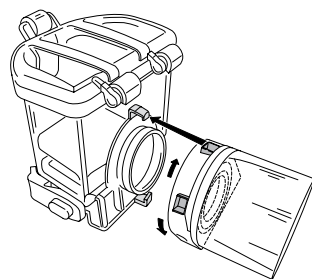
- The atomiser and accessories must be cleaned and disinfected before they are used for the first time. For

information about this, see “Cleaning and disinfection” on page 17.


Assembly

Remove the unit from the packaging.

- If not already fitted, fit the mesh atomiser (3) onto the neck of the container (2), and lock into place by turning it clockwise until the silver-coloured contact electrodes are facing downwards.

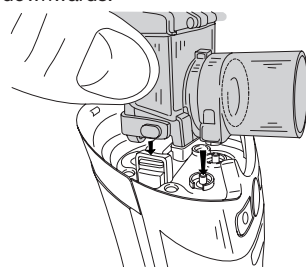
- Place the container with fitted mesh firmly onto the housing (6) or base of the device, so that you hear it click into place on the left and right.
- Insert the batteries.

Inserting the batteries

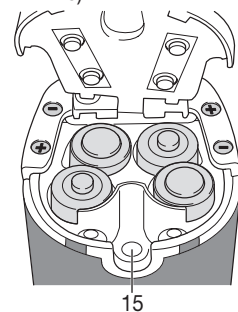- To open the battery compartment (7) on the underside of the device, apply light pressure and pull the battery compartment cover in the direction of the arrows, then fold the lid up.
- Insert four batteries (type AA LR6).
- Ensure that the poles of the batteries are touching their respective poles in the device (see illustration on the bottom of the casing).
- Fold the battery compartment lid down again and push it back in place, in the opposite direction to the arrows.

i Note

- When changing batteries, ensure that the medicine container is completely empty, otherwise there is a risk of leaks.
- When the orange LEDs (4) light up, replace all four batteries, because spraying becomes much less effective when the batteries are weak.

- You can operate the device for approx. 180 minutes with new alkaline batteries.
- This is sufficient for 5–9 uses of saline solution (depending on the dose and duration of each use).

Operating the device with the power pack

Only connect the power pack (13) to the mains voltage listed on the type plate.

- Open the battery compartment lid to connect the power pack (13).
- Insert the power pack connector (8) into the connection socket provided (15), and the power pack (13) fully into a suitable socket.

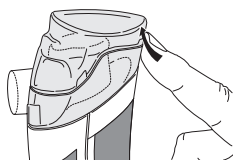
i Note

- Ensure that there is a socket near to where the unit will stand.
- Lay the mains cable in such a way that no one can trip over it.
- To disconnect the nebuliser from the mains after use, first switch off the device and then remove the plug from the socket.
- Do not remove the batteries when operating the device with the power pack (13).
- The power pack (13) cannot be used for charging batteries.

7. Operation

1. Preparing the atomiser

- For reasons of hygiene, it is imperative that the atomiser and accompanying accessories are cleaned and disinfected after every use.


If the therapy requires that various different medicines be inhaled in succession, ensure that the atomiser is rinsed after every use. For details, see Cleaning and disinfection on page 17.

- Remove the cover (1) from the device.

2. Filling the atomiser

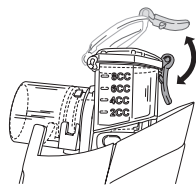- Open the medicine container (2) by opening the clip, and fill the medicine container with an isotonic saline solution or medicine. Do not overfill! The maximum recommended fill amount is 8 ml.
- Use medication only on the advice of a physician, and check the appropriate inhalation duration and quantity for your needs.
- If the recommended dose of the medicine is less than 2 ml, top up this quantity to at least 4 ml using

only isotonic saline solution. Viscous medicines may also require dilution. Always follow the instructions of your doctor.

3. Closing the atomiser

- Close the lid of the medicine container (2) and click the clip in place, and then refit the cover (1).

4. Fitting accessories

- When the required accessories (mouthpiece, adult mask or child mask) have been attached to the device, attach the device firmly to the assembled medicine container (2).
- The adult mask and child mask must be fitted using the connecting piece (12) (adapter).
- Now place the device to your mouth and close your lips firmly around the mouthpiece. When using either of the masks, place it over the nose and mouth.
- Start the device using the on/off button (5).
- The spray mist coming from the device and the illuminated blue LEDs (4) show that the device is working perfectly.

5. Inhaling correctly

• Breathing technique

The right breathing technique is important to ensure that the particles are distributed as widely as possible into the air ways. So that they can enter the air ways and lungs, you must inhale slowly and deeply, hold your breath briefly (5 to 10 seconds) and then exhale quickly.

- You should only use nebulisers to treat respiratory disorders if advised to do so by your doctor, who will recommend the medicine, dose and application for inhalation therapy.
- Certain medicines are only available on prescription.

i Note

You should hold the device as upright as possible. Tilting it slightly will not, however, affect its function, since the container is leakproof. To ensure that the device functions correctly, make sure that you do not tilt it more than 45° in any direction and that the medicine is in contact with the mesh while in use.

! Important

Essential oils, cough medicines, solutions designed for gargling, and drops for application to the skin or for use in steam baths are wholly unsuitable for inhalation using a nebuliser. These substances are often highly viscous and can impair the correct function of the device and hence affect the effectiveness of its application in the long term.

In the case of oversensitivity of the bronchial system, medicines containing essential oils can sometimes cause acute bronchospasm (a sudden, cramp-like tightening of the bronchi accompanied by breathlessness). Always ask your doctor or pharmacist for advice!

6. End of inhalation

- After treatment, switch off the device using the on/off button (5).
- The LEDs (4) go out.
- Once the inhalant has been fully sprayed out, the device switches off automatically.
For technical reasons, a small amount may still remain in the medicine container (2).
- Do not use it.
- If applicable, disconnect the power pack (13) from the mains.

7. Clean the device

For information about this, see “Cleaning and disinfection” on page 17.

8. Cleaning and disinfection

Warning

Observe the following hygiene instructions in order to avoid any health risks.

- The nebuliser and accessories are intended for multiple use. Please note that cleaning and hygienic reconditioning requirements differ according to the different areas of application.

Note

- Do not clean the mesh or accessories with brushes or similar, as this may cause irreparable damage, and this will mean that the intended result of the treatment is no longer guaranteed.
- For additional requirements regarding the necessary hygienic preparations (hand washing, handling of medicines or inhalation solutions) in high-risk groups (e.g. cystic fibrosis patients), contact your doctor.

Cleaning

The mesh atomiser and container, as well as accessories such as mouthpiece, mask, connecting piece, etc. must be cleaned with hot water after each use. Carefully dry all components using a soft cloth. Once all components are fully dry, reassemble the atomiser and place the components in a dry, sealed container or carry out disinfection.

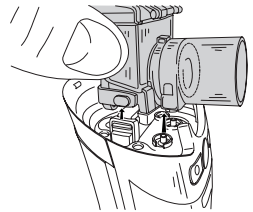
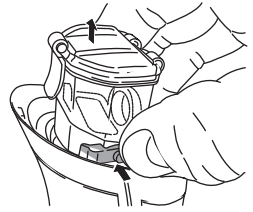
- Remove the mouthpiece and any accessories from the atomiser.
- Add 6 ml of clean water to the medicine container (2) and switch on the device to first remove any medicine residue from the mesh. (Please spray the entire liquid).
- Remove the batteries.

- Remove the detachable parts from the device, e.g. mesh atomiser or medicine container and cover.


Disassembly

To continue with cleaning its individual components, disassemble the atomiser (2, 3):

- Remove the medicine container (2) and mesh atomiser (3) by pressing both brackets inwards and drawing the unit out.

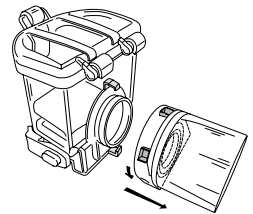- Remove the mesh atomiser (3) from the medicine container (2) by turning it anticlockwise and drawing it out.
- The nebuliser is later reassembled in the reverse order.

Important

Never hold the complete device under running water to clean it.

When cleaning, ensure that all residues are removed. Never use any substances that may be poisonous when in contact with the skin or mucous membranes, or when swallowed or inhaled.

If required, you can clean the device casing with a lightly moistened cloth that has been soaked in mild soapy water.

- If there is any residue from medical solutions or contamination on the silver-coloured contacts of the device or atomiser, clean with a cotton bud moistened with ethyl alcohol.
- Clean the outside of the mesh and outside of the medicine container with a cotton bud dampened with ethyl alcohol.

Important

- Before you clean the unit, always switch it off, unplug it and let it cool off.
- Do not use abrasive detergents and never immerse the unit in water.

- Ensure that no water penetrates inside the unit!
- Do not put the device or accessories in a dishwasher.
- Do not touch the unit with wet hands while it is plugged in; do not allow any water to be sprayed onto the unit. The unit must be operated only when it is completely dry.
- Do not spray any liquids into the ventilation slots. Any liquid that penetrates into the unit can damage the electrical parts or other components of the nebuliser and impair the function of the device.

Drying

- Carefully dry all components using a soft cloth.
- Shake the mesh atomiser (3) lightly backwards and forwards (5 to 10 times), so that the water inside the mesh is removed from the small holes.
- Place the individual components on a dry, clean and absorbent surface and allow to dry completely (for at least 4 hours).

Note

Ensure that all parts are thoroughly dried after cleaning, otherwise there is an increased risk of bacterial growth. Once all components are fully dry, reassemble the nebuliser and place the components in a dry, sealed container.

Ensure that the mesh atomiser (3) has dried completely by shaking it. Otherwise, nebulisation may not function properly after reassembling the device.

If this is the case, disassemble the mesh atomiser (3) again and shake it so that the water can escape. Once it has been reassembled, the nebuliser should function properly again as usual.

Material resistance

- When choosing a cleaning or disinfecting agent, note the following: Only use mild cleaning or disinfectant agents in the quantities recommended by the manufacturer.
- As with all plastic components, frequent use and cleaning of the atomiser and the accessories can lead to a certain amount of wear. Over time, this can alter the aerosol properties and may eventually affect the efficiency of therapy. We therefore recommend that you replace the atomiser and accessories at least once a year.

Storage

- Do not store the nebuliser in a damp atmosphere (e.g. in the bathroom) or transport it together with damp objects.
- Store and transport the nebuliser away from direct sunlight.

Troubleshooting

Problem/Question	Possible Cause/Remedy
The nebuliser produces little or no aerosol.	1. Insufficient medicine in the atomiser.
	2. The atomiser is not held in an upright position.
	3. The medicine used is unsuitable for spraying (e.g. too thick, viscosity must be less than 3). The medicine solution should be specified by the doctor.

Disinfection

Follow the steps listed below carefully in order to disinfect your atomiser and the accessories. It is advisable to disinfect the individual components after the last time they are used each day at the latest.

You can disinfect the IH 50 and its accessories in two different ways, either with ethyl alcohol or in boiling water.

First clean the atomiser and the accessories as described under "Cleaning". You can then continue with disinfection.

Important

Ensure that you do not touch the mesh (3), as this may destroy it.

Disinfection with ethyl alcohol (70-75%)

- Pour 8 ml ethyl alcohol into the medicine container (2). Close the container. Leave the alcohol in the container for at least 10 minutes.
- For better disinfection, shake the unit gently from time to time.
- Afterwards, pour the ethyl alcohol out of the container.
- Repeat this process, using water this time.
- Place the medicine container (2) in a position that allows you to apply a few drops of ethyl alcohol to the mesh (3). Allow the alcohol to work for 10 minutes.
- Finally, clean all parts again under running water.

Disinfection with boiling water

- The atomiser should be disassembled as described and, together with the mouthpiece, placed in boiling hot water for 15 minutes. Avoid the parts coming into contact with the hot base of the pot.
- You can also disinfect the atomiser and the mouthpiece with a standard vaporiser. Observe the manufacturer's instructions for use for the vaporizer when doing this.
- The atomiser must not be placed in the microwave.

Note

The face masks and air hose must not be placed in hot water!

- We recommend disinfecting the masks using a retail disinfectant.

Problem/Question	Possible Cause/Remedy
Output is too low.	1. The batteries are flat. Replace the batteries or connect the power supply unit and try again.
	2. There are air bubbles inside the medicine container which are preventing the medicine from coming into continuous contact with the mesh. Please check and remove any air bubbles.
	3. Particles on the mesh are impeding output. To remove the particles, fill the nebuliser with 2 to 3 drops of vinegar and 3 to 6 ml of water, then nebulise this solution completely. Do not inhale this spray and ensure that you clean and disinfect the medicine container afterwards (see page 17). If the output does not improve, replace the mesh.
	4. The mesh is worn out.
Which medicines are suitable for inhalation?	Only the doctor can advise you which medicine to use to treat your condition. Consult your doctor. With the IH 50, you can atomise medicines with a viscosity lower than 3.
Some inhalation solution remains in the nebuliser.	This is normal and occurs for technical reasons. Stop inhaling as soon as you can hear a marked difference in the sound made by the atomiser, or when the device switches off automatically due to insufficient inhalant.
What special steps should be taken for babies and children?	<p>1. In babies, the mask should cover the mouth and nose to guarantee effective inhalation.</p> <p>2. In children, the mask should also cover the mouth and nose. Nebulisation next to a sleeping person is not suitable because insufficient medicine reaches the lungs.</p> <p>Note: Children should only use the device with help and under supervision of an adult. Never leave a child alone with the nebuliser.</p>
Inhalation with the mask takes longer.	This is for technical reasons. Less medicine is inhaled per breath through the mask holes than using the mouthpiece. The aerosol is mixed with ambient air through the holes in the mask.
Does each user need their own accessories?	This is absolutely necessary for hygiene reasons.


9. Technical specifications

Dimensions (LxWxH)	72 x 59 x 143 mm
Weight	238 g incl. batteries
Battery operation	4 x 1.5V type AA, Mignon (LR6)
Volume	max. 8 ml
Medicine flow	approx. 0.25 ml/min.
Oscillation frequency	100 kHz
Housing material	ABS / TPE
Mains connection	100–240V ~, 50–60 Hz; 0,15 A
Operating conditions	Temperature: +5°C – +40°C Relative humidity: <80 % non-condensing
Storage and transport conditions	Temperature: -10°C – +45°C Relative humidity: <80 % non-condensing

Subject to technical modifications.
Wearing parts are not covered by the guarantee.

Spare parts

Designation	Material	REF
Mouthpiece	PP	162.658
Adult mask	PVC	162.712
Child mask	PVC	162.659
Mesh atomiser with medicine container	PC	162.711
Power pack	PC+ABS	071.18
Connecting piece	PC	162.665
Yearpack (Mouthpiece, Adult mask, Child mask, Mesh atomiser, Connecting piece)	PP, PVC, EVA, PET	603.05

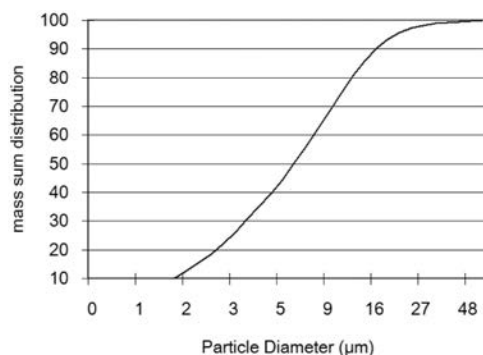All measurements were obtained with a sodium chloride solution using a laser diffraction method.

This diagram may not be applicable for suspensions or highly viscous medicines. More information can be obtained from the relevant medicine manufacturer.

Note

If the unit is used outside of the specifications, proper function is no longer guaranteed! We reserve the right to make technical changes to improve and further develop the product.

This device and its accessories comply with European standards EN60601-1 and EN60601-1-2, as well as EN13544-1, and is subject to special safety measures in terms of electromagnetic tolerance. Note that portable and mobile RF communication equipment can affect this unit. More details can be requested from the stated Customer Service address or found at the end of the instructions for use. The unit conforms to the requirements of the European Directive for Medical Products 93/42/EC, the MPG (German Medical units Act).

ELECTROMAGNETIC COMPATIBILITY

- The device complies with current specifications with regard to electromagnetic compatibility and is suitable for use in all premises, including those designated for private residential purposes. The radio frequency emissions of the device are extremely low and in all probability do not cause any interference with other devices in the proximity.
- It is recommended that you do not place the device on top of or close to other devices. Should you notice any interference with other electrical devices, move the unit or connect it to a different socket.
- Radio equipment may affect the operation of this device.

10. Disposal

The appliance should be disposed of according to Regulation **2002/96/EC - WEEE** (Waste Electrical and Electronic Equipment). In case of  queries, please contact the municipal authorities responsible for waste disposal in your area.

Used batteries do not belong in the household waste. You are legally obliged to dispose of the batteries. Dispose of them via your specialist electrical supplier or local collecting point for recyclable waste.

Note: Batteries containing pollutant substances are marked as follows:

Pb = Battery contains lead,

Cd = Battery contains cadmium,

Hg = Battery contains mercury.

The batteries for this device do not contain any toxic substances.

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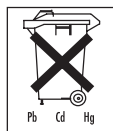
Table 1

For all ME EQUIPMENT and ME SYSTEMS

Guidance and manufacturer's declaration – electromagnetic emissions		
The IH 50 is intended for use in the electromagnetic environment specified below. The customer or the user of the IH 50 should assure that it is used in such an environment.		
Emissions	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The IH 50 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The IH 50 is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class C	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Complies	

Table 2

For all ME EQUIPMENT and ME SYSTEMS

Guidance and manufacturer's declaration – electromagnetic immunity			
The IH 50 is intended for use in the electromagnetic environment specified below. The customer or the user of the IH 50 should assure that it is used in such an environment.			
Immunity test	EN 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines	± 2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) and neutral	± 1 kV line(s) and neutral	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % U_T (>95 % dip in U_T) for 0.5 cycle 40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30 % dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5 s	<5 % U_T (>95 % dip in U_T) for 0.5 cycle 40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30 % dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the IH 50 requires continued operation during power mains interruptions, it is recommended that the IH 50 be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	Not applicable	Not applicable

NOTE: U_T is the a.c. mains voltage prior to application of the test level.

Table 3

For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING

Guidance and manufacturer's declaration – electromagnetic immunity			
The IH 50 is intended for use in the electromagnetic environment specified below. The customer or the user of the IH 50 should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3V _{rms} 150 kHz to 80 MHz	3V _{rms}	Portable and mobile RF communications equipment should be used no closer to any part of the IH 50, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance: $d = 1,2 \sqrt{P}$ $d = 1,2 \sqrt{P}$ 80 MHz to 800 MHz $d = 2,3 \sqrt{P}$ 800 MHz to 2,5 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC 61000-4-3	3V/m 80 MHz to 2,5 GHz	3V/m	

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the IH 50 is used exceeds the applicable RF compliance level above, the IH 50 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the IH 50.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.

Table 4
For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING

Recommended separation distances between portable and mobile RF communications equipment and the IH 50

The IH 50 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the IH 50 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the IH 50 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d = 1,2 \sqrt{P}$	80 MHz to 800 MHz $d = 1,2 \sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3 \sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be determined using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.